


Always at hand and easy to use: Wipes for wipe disinfection

Dürr Dental disinfection and cleaning wipes

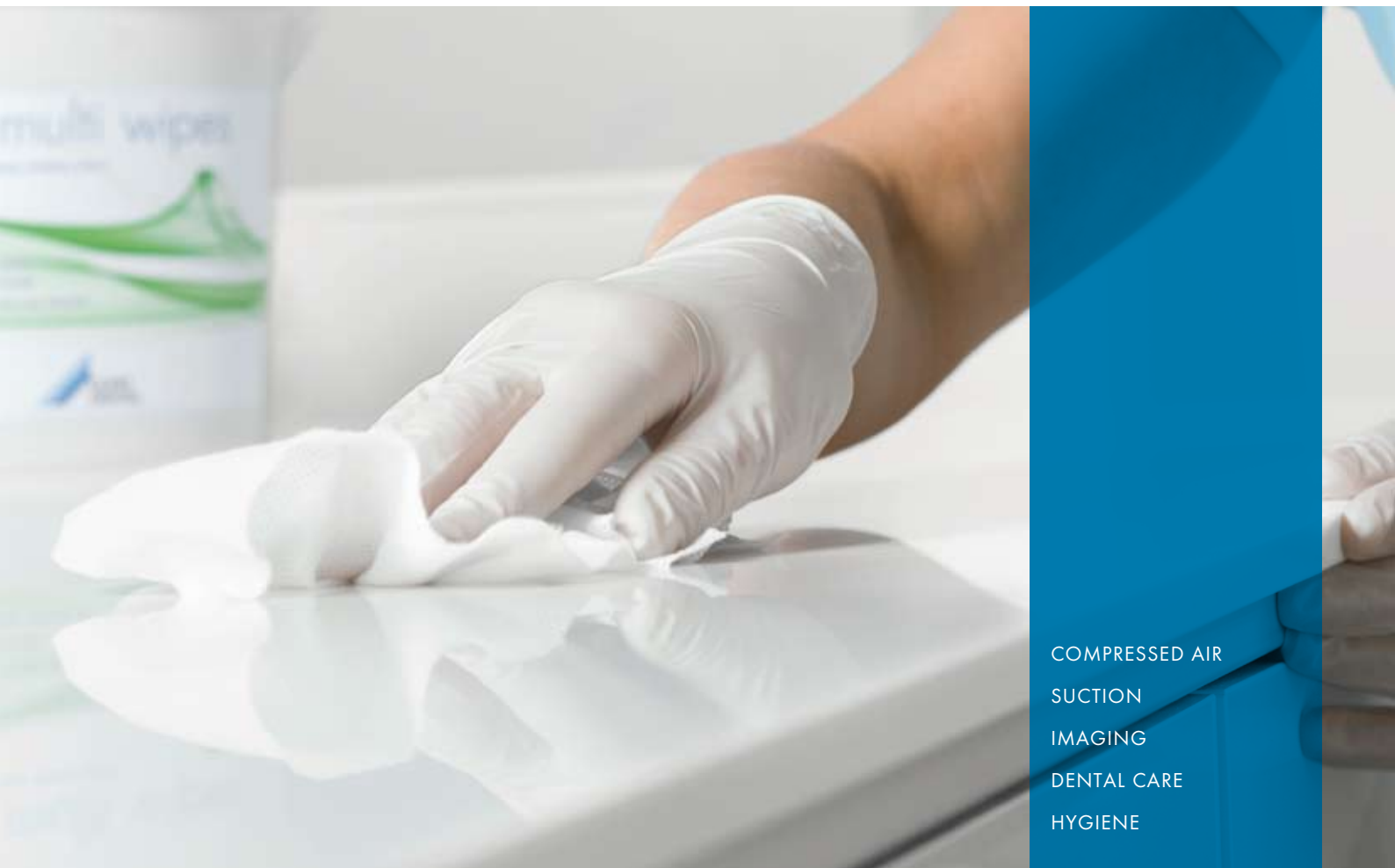

COMPRESSED AIR
SUCTION
IMAGING
DENTAL CARE
HYGIENE

The right wipe for each purpose



Ready-to-use disinfectant wipes are very practical and have come to play an important role in every modern surgery. They are particularly suited for the disinfection and cleaning measures after every patient or for the quick disinfection of the floor between treatments, and generally for all occasions when work needs to be swift. Wipes should be the product of choice whenever surfaces need to be prepared quickly.

As they can cover a wide range of applications, the wipes offered by Dürre Dental come in different sizes, materials, ranges of action as well as different reaction times and for different purposes – even including unsoaked wipes, allowing to create one's own ready-to-use wipes with the preferred disinfectant.

This is what counts:

- For every application the right size and material
- Always at hand and ready to use
- Exceptional material compatibility
- Wide range of action
- Optimum reaction time depending on the purpose
- Modern effective ingredients, high quality standards
- Products with VAH/DGHM* certification, complying with all applicable standards

*in accordance with VAH/DGHM: German associations of scientific experts that dedicate themselves to the overall promotion and medical research in the area of infectiology and microbiology.

As versatile and individual as you require



For trays and deposit surfaces: FD 300 top wipes

Alcohol-free, fully virucidal ready-to-use disinfectant and cleaning wipes with ideal size (17.5 x 28 cm) for wipe disinfection with immediate effect (reaction time 2 minutes), in particular for furnishing surfaces and treatment units. The comprehensive range of action, the clever structure for convenient handling and the cleaning properties are "top of class" – making the FD 300 top wipes the first choice in this field. Also suited for alcohol-sensitive surfaces.



For particularly large surfaces: FD 312 wet wipes

Their size (22 x 42 cm) makes the alcohol-free, ready-to-use FD 312 wet wipes exceptionally well suited for the disinfection and cleaning of large surfaces (up to 8 m²) and objects. The tear-resistant honeycomb structure provides maximum cleaning power, whether for surgery inventory, appliances or floor surfaces. The package is recloseable and prevents drying out.



For small surfaces and medical devices: FD 350 disinfectant wipes

Alcoholic disinfectant wipes for the wipe disinfection and cleaning of medical devices, with a reaction time of 1 minute. Ideal and economical for smaller objects thanks to their size: e.g. handpieces and elbows, sleeves of intraoral films, etc. Can be used immediately, free of residues! Available in 3 different fragrances: Classic, Flower and Lemon.



For universal applications: FD multi wipes

The FD multi wipes allow you to determine the type and effectiveness of the disinfectant yourself by wetting or soaking the wipes with your preferred surface disinfectant: e.g. with the usable solutions of the concentrated liquids FD 300 or FD 312 as well as the preparations FD 322, FD 333 or FD 366 sensitive. Furthermore, the wipes can also be used for dry applications (extra large: 20 x 30 cm). Ideal for cost-effective application tailored to your specific needs. Please note: Dry cloths from the supermarket frequently do not go well with disinfectants (e.g. fuzzing).



Facts and data

	FD 300 top wipes	FD 312 wet wipes	FD 350 disinfectant wipes	FD multi wipes
Ready to use	■	■	■	
Alcohol-free	■	■		*)
Virucidal to some extent (enveloped viruses, incl. HBV, HCV and HIV)	■	■	■	*)
Virucidal (effective against all viruses)	■			*)
Dimensions (cm)	17.5 x 28 cm	22 x 42 cm	14 x 22 cm	20 x 30 cm
Grammage	50 g/m ²	100 g/m ²	28 g/m ²	50 g/m ²
Pack sizes	Can with 50 wipes 4 refill packs per box	Pack with 15 wipes	Can with 90 wipes 12 refill packs per box	Box with 180 wipes 4 refill packs per box
Reaction time	2 min	60 min	1 min	*)
Fragrances			in Classic, Flower, Lemon as preferred	

*1) depending on the disinfectant used

Wipe sizes

